

Baxter

Hillrom Centrella

SMART+ BED

SKILLS CHECKLIST



Staff Signature: _____

Trainer's Signature: _____

Date: _____

Perform the following.

COMPLETED

Bed Operations

1. Demonstrate how to attach the mattress to the frame of the bed	
2. Demonstrate how to ensure loose-fitting sheets are attached to the mattress	
3. Review the two power cords: grey = power, white = accessory outlet	
4. Explain how bed indicates the communication/call light is not connected	
5. Demonstrate the IntelliDrive Power Transport system and brake and steer mechanism	
6. Identify the built-in line managers	
7. Demonstrate CPR function and explain what happens to the mattress	
8. Demonstrate the Boost function	
9. Demonstrate Trendelenburg and Reverse Trendelenburg	
10. Demonstrate Lockout controls	
11. Locate patient accessible siderail features: USB port, siderail storage, and urinal holder	
12. Discuss illuminated siderail grip. Explain meaning when light is blue, amber, flashing amber	
13. Discuss verbal alerts	
14. Identify attachment points for restraint	
15. Identify the hip indicator label for patient placement in the bed	
16. Locate and explain activation of the night light	
17. Identify the foley/drainage bag holder	
18. Explain the foley position limit. Demonstrate how to turn it on and how to recognize it is active	
19. Identify and explain the SafeView+ icons (at foot-end of the bed and on the floor). Explain meaning of green, flashing amber, and blue floor projections	

GCI TouchScreen

20. Identify the head of bed indicators	
21. Demonstrate how to set the <30 degree head of bed limit	
22. Demonstrate the FlexAfoot feature	
23. Demonstrate the stand assist control	
24. Demonstrate the chair position	
25. Demonstrate the bed flat control	
26. Locate the Help Screens and Service Wrench	
27. Define the three-levels of bed exit sensitivity	

SKILLS CHECKLIST

Perform the following.

COMPLETED

Bed Exit

28. Demonstrate how to set bed exit alert	
29. Demonstrate how to silence the bed exit alert	
30. Demonstrate how to resume bed exit alert	

Scale

31. Explain difference between the scale “new patient zero” and “same patient re-zero”	
32. Demonstrate how to zero scale	
33. Demonstrate how to weigh patient, switch between lbs and kg, and recall weight after saving	
34. Explain how you can visualize that the bed has been zeroed and is ready for a new patient	

Choose your surface:

Baxter

Hillrom Centrella

SMART+ BED

SKILLS CHECKLIST

Perform the following.

COMPLETED

Hillrom Centrella max+ Surface

1. Discuss how Focused Advanced Microclimate Technology works	
2. Discuss how weight based pressure redistribution works	
3. Demonstrate TurnAssist, Max Inflate and Comfort Adjust	
4. Demonstrate Continuous Lateral Rotation and how to adjust rotation levels and hold times.	

Baxter reserves the right to make changes without notice in design, specifications and models. The only warranty Baxter makes is the express written warranty extended on the sale or rental of its products.

Baxter, Advanced Microclimate, Boost, Centrella, FlexAfoot, Hillrom, IntelliDrive and SafeView are trademarks of Baxter International, Inc. or its subsidiaries.



Perform the following.

Hillrom Centrella max Surface

1.	Discuss how Focused Advanced Microclimate Technology works	
2.	Discuss how weight based pressure redistribution works	
3.	Demonstrate TurnAssist, Max Inflate and Patient Comfort Controls	

US-CS3-230063 v1 11/23



Perform the following.

Hillrom Centrella pro+ Surface

1. Discuss how Focused Advanced Microclimate Technology works, how to turn on/off	
2. Discuss how pressure redistribution works	

US-CS3-230063 v1 11/23

Perform the following.		COMPLETED
Hillrom Centrella pro Surface		
1.	Discuss how pressure redistribution works	



Perform the following.

COMPLETED

1. Discuss how pressure redistribution works