



# Nipro Cellentia™-H Priming Guide



**STEP 1**  
Place dialyzer in the holder with the arterial end up.



**STEP 4**  
Rotate dialyzer to venous end up and connect the **blue venous line** to the dialyzer. Prime the dialyzer at 100ml/min and continue as per clinic's policy and procedure.



**STEP 2**  
Connect the bloodlines as per manufacturer's instructions. Prime the bloodlines as per clinic's policy and procedure.



**STEP 5**  
Continue priming and recirculation process until 1000mL of saline has passed through the dialyzer. Administration of saline may be determined by facility policy and procedure.



**STEP 3**  
Once all air is purged, connect the **red arterial line** to the dialyzer.

**STEP 6**  
Once all steps have been completed as per clinic's policy and procedure, the dialyzer is ready for use in treatment.